

PERÖXAN[®]

Naghsh-e-Jobbe Pars

www.peroxan.com





**MEDICAL
SURFACE
SANITIZER**



**Fast-acting
disinfectant**



**Kills up to %99.99
of Viruses,
Bacteria and Fungi**



**UNICERT
GMP**



► **GMP**

Universal GmbH UNICERT

► **CE**

European conformity

► **ISO 9001:2015**

Quality management

► **ISO 22000:2005**

► **ISO 14001:2004**

Environment management standard

► **OHSAS 18001:2008**

Occupational health & safety
advisory services

► **ISO 10002:2004**

Quality management -customer
satisfaction

Naghsh-e-jobbe Pars Co. IRAN-Semnan

Tel: +98 23 33653459 Fax: +98 23 33653487

SMS Customer Service:10000822391815

www.peroxan.com Email:njpc@yahoo.com

TETRASEPT F®

MEDICAL SURFACE SANITIZER

**Ready-to-use and fast-acting disinfectant for
hygienic and medical surfaces**

+ **Uses:**

Ready-to-use and fast-acting disinfectant for surfaces with a broad spectrum of antimicrobial effective on gram-positive and gram-negative bacteria, Mycobacteria, fungi, molds and coated and uncoated viruses.

This product is used for quick disinfection of surfaces, and equipment of hospitals, clinics, doctors' and dentists' offices.

+ **Benefits:**

- Effective on a wide range of microorganisms
- contains quaternary ammonium compounds and various alcohols to accelerate, persistence and increase antimicrobial properties.
- Ready-to-use solution
- Compatible with different surfaces
- Eliminates unpleasant odors
- Easy-to-use product without need to rinse

+ **Product appearance:**

Clear colorless liquid

+ **Active ingredients:**

Ethanol, Isopropanol, Benzalkonium chloride, EDTA, Fragrance, Deionized water

+ **Direction for use:**

At first, clean the surface from obvious contaminants such as blood, body secretions, etc. with water, then spray the surface of the equipment and disinfect it. The minimum contact time with the disinfectant solution is 2 minutes.

+ **Warnings:**

- It is a flammable solution
- Keep away from sunlight and heat sources
- Avoid contact with skins, in case of contact with skin immediately wash skin with water and see a doctor.
- Keep out of reach of children and food.

+ **Packing:**

Fast-acting surface disinfectant solution in polyethylene containers are available in 250, 350 and 500 Milliliter and 1, 5, 10, 20 Liter volumes.


+ **Other information:**

Storage conditions: Store between 5-35 °C in closed container and keep away from the direct sunlight, flammable materials and heat.



Naghsh-e-Jobbe Pars

Health for all- Peroxan, Messenger of Health

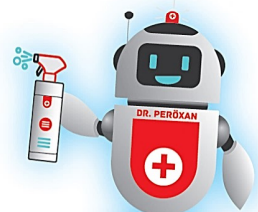



HOME &
ELECTRONICS
SURFACE
SANITIZER



Fast-
acting
disinfectant



Kills up to %99.99
of Viruses,
Bacteria and Fungi



► GMP

Universal GmbH UNICERT

► CE

European conformity

► ISO 9001:2015

Quality management

► ISO 22000:2005

► ISO 14001:2004

Environment management standard

► OHSAS 18001:2008

Occupational health & safety
advisory services

► ISO 10002:2004

Quality management -customer
satisfaction

Naghsh-e-jobbe Pars Co. IRAN-Semnan

Tel: +98 23 33653459 Fax: +98 23 33653487

SMS Customer Service:10000822391815

www.peroxan.com Email:njpc@yahoo.com

TETRASEPT G[®]

SURFACE SANITIZER

HOME & ELECTRONICS SURFACE SANITIZER

Ready-to-use Alcoholic disinfectant solution for
general surfaces without need to rinse

+ Uses:

- To clean and disinfect surfaces in direct contact with food such as meat grinder and surfaces for chopping raw materials such as vegetables, meat, chicken, fish, etc. or surfaces indirect contact with food such as dining table, inside surfaces in refrigerator and Microwaves, Knobs and pulls are used.

Electronic devices

- Made of high quality raw materials
- Appropriate for different surfaces
- A safe product for daily use
- Easy-to-use product
- Eliminates unpleasant odors
- No need to dilute
- General surface disinfectant solution with a wide range of antimicrobials
- Effective on gram-positive and gram-negative bacteria, mycobacteria, fungi and Coated and uncoated viruses

+ Benefits:

- Fast-acting disinfectant
- without need to rinse with an amazing effect on microorganisms in our living environment and brings us a germ-free surface

+ Product appearance:

Clear colorless liquid

+ Active ingredients:

Ethanol, Benzalkonium chloride, Deionized water-Fragrance

+ Direction for use:

Spray some of the solution (as needed) on the surface from a distance of about 30 cm.

In the case of food-related surfaces, they should be rinsed after about 2 minutes.

In the case of non-food surfaces such as the dining table and child seat, the disinfectant solution should be sprayed on the surface and after about 2 minutes to prevent surface from opacity should be wiped with a clean, dry napkin.

+ Warnings:

- It is a flammable solution
- Keep away from sunlight and heat sources
- Avoid contact with skins, in case of contact with skin immediately wash skin with water and see a doctor.
- Keep out of reach of children and food.

+ Packing:

Fast-acting surface disinfectant solution in polyethylene containers are available in 250, 350 and 500 Milliliter and 1, 5, 10, 20 Liter volumes.

+ Other information:

Storage conditions: Store between 5-35 °C in closed container and keep away from the direct sunlight, flammable materials and heat.



Naghsh-e-Jobbe Pars

Health for all- Peroxan, Messenger of Health





**MEDICAL
SURFACE
SANITIZER**



Fast-acting
& Lasting
disinfectant



**Kills up to %99.99
of Viruses,
Bacteria and Fungi**



► **GMP**
Universal GmbH UNICERT

► **CE**
European conformity

► **ISO 9001:2015**
Quality management

► **ISO 22000:2005**
► **ISO 14001:2004**
Environment management standard

► **OHSAS 18001:2008**
Occupational health & safety
advisory services

► **ISO 10002:2004**
Quality management -customer
satisfaction

Naghsh-e-jobbe Pars Co. IRAN-Semnan
Tel: +98 23 33653459 Fax: +98 23 33653487
SMS Customer Service:10000822391815
www.peroxan.com Email:njpc@yahoo.com

TETRASEPT S[®]

SURFACE SANITIZER

MEDICAL SURFACE SANITIZER

**Fast-acting & lasting alcoholic disinfectant for
medical and dental surfaces and equipment**
Ready-to-use product

+ Uses:

This is a ready-to-use product which is an alcohol-based solution for quick disinfection of small environmental surfaces and medical equipment that require quick and powerful disinfection. Includes surfaces in contact with patients, such as patients' beds, dental units, medical equipment and surfaces, and equipment areas in clinics and laboratories.

Surfaces of metal medical diagnostic tools such as otoscopes, laryngoscope handle and nursing station surfaces. Effective on coated and uncoated viruses, bacteria, fungi, and mycobacterium tuberculosis.

+ Benefits:

This solution is quick and effective. Due to the presence of ethanol in the product and its effectiveness in 2 minutes, it has lasting effects. It has a wide range of effects and it is so compatible with all hospital surfaces. There are no toxic substances in its compounds.

+ Product appearance:

Clear colorless liquid

+ Active ingredients:

Ethanol, Isopropanol, Benzalkonium chloride, EDTA, Fragrance, Deionized water

+ Direction for use:

The solution does not need to be diluted and is ready to use.

First, clean and dry the area and spray the disinfectant on the surface. This solution will have its antimicrobial activity in 2 minutes.

+ Warnings:

- The solution is not compatible with Plexi glass surfaces.
- Avoid spraying it on the flame
- Avoid eye contact and swallowing
- Avoid spraying the solution on electronic devices
- If swallowed, do not provoke vomiting and just see a doctor

+ Packing:


Fast-acting surface disinfectant solution in polyethylene containers are available in 250, 350 and 500 Milliliter and 1, 5, 10, 20 Liter volumes.

+ Other information:

- It is a flammable solution
- Keep away from sunlight and heat sources
- Storage conditions: Store between 5-35 °C in closed container

Naghsh-e-Jobbe Pars

Health for all-Peroxan, Messenger of Health

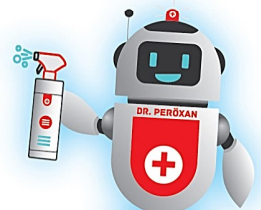



CONCENTRATED
MEDICAL
SURFACE
SANITIZER



**Kills up to %99.99
of Viruses,
Bacteria and Fungi**



► **GMP**
Universal GmbH UNICERT

► **CE**
European conformity

► **ISO 9001:2015**
Quality management

► **ISO 22000:2005**

► **ISO 14001:2004**
Environment management standard

► **OHSAS 18001:2008**
Occupational health & safety
advisory services

► **ISO 10002:2004**
Quality management -customer
satisfaction

Naghsh-e-jobbe Pars Co. IRAN-Semnan
Tel: +98 23 33653459 Fax: +98 23 33653487
SMS Customer Service:10000822391815
www.peroxan.com Email:njpc@yahoo.com

TETRAMO S[®]

SURFACE SANITIZER

MEDICAL SURFACE SANITIZER

**Concentrated solution for disinfection
of general and hygienic surfaces**

+ Uses:

- Concentrated disinfectant for surfaces with a broad spectrum of antimicrobial effective on gram-positive and gram-negative bacteria, Mycobacteria, fungi, molds and coated and uncoated viruses.
- This product is soluble in water with different percentages for disinfection of all surfaces in hospitals, clinics, doctors' and dentists' offices.

+ Benefits:

- Ability to clean along with disinfection
- The clean diluted solution lasts up to 7 days
- Low and effective foam in the presence of water hardness
- Effective on a wide range of bacteria, fungi and viruses
- Without annoying fumes
- High durability and stable effect

+ Product appearance:

Clear yellow liquid

+ Active ingredients:

Didecyltrimethylammonium chloride, Benzalkonium chloride, EDTA, Deionized water

+ Direction for use:

Dilute the solution to a concentration of 2% (Dissolve 20 ml of the solution in 980 ml of plain water). Squeegee the surfaces with the solution. Wait 30 minutes in order to disinfect the area.

For smaller surfaces, can spray 2% diluted solution on the clean cotton napkin and disinfect the surfaces with it.

+ Warnings:

- Avoid contact with skins, in case of contact with skin immediately wash skin with water and see a doctor.
- Keep out of reach of children and food.

+ Packing:

Fast-acting surface disinfectant solution in polyethylene containers are available in 250, 350 and 500 Milliliter and 1, 5, 10, 20 Liter volumes.

+ Other information:

- It is a flammable solution
- Keep away from sunlight, flame and heat sources.
- Storage conditions: Store between 5-35 °C in closed container



Naghsh-e-Jobbe Pars

Health for all- Peroxan, Messenger of Health





**ALCOHOLIC
HAND
ANTISEPTIC
GEL**



**ALCOHOLIC
GEL**



**Kills up to %99.99
of Viruses,
Bacteria and Fungi**



► **GMP**
Universal GmbH UNICERT

► **CE**
European conformity

► **ISO 9001:2015**
Quality management

► **ISO 22000:2005**

► **ISO 14001:2004**
Environment management standard

► **OHSAS 18001:2008**
Occupational health & safety
advisory services

► **ISO 10002:2004**
Quality management -customer
satisfaction

Naghsh-e-jobbe Pars Co. IRAN-Semnan
Tel: +98 23 33653459 Fax: +98 23 33653487
SMS Customer Service:10000822391815
www.peroxan.com Email:njpc@yahoo.com

DAS HG[®]

ALCOHOLIC ANTISEPTIC HAND GEL

**Ready-to-use alcoholic hand antiseptic gel with
softener without need to rinse**

+ Uses:

This product has a broad spectrum of antimicrobial activity for a variety of gram-positive and gram-negative mycobacteria, fungi, coated and uncoated viruses.

+ Benefits:

- Fast effect without need to rinse
- Compatible with hand skin with persistent effect on the hands
- With special formulation and has quaternary ammonium compounds with softener and moisturizing materials to increase effectiveness and prevent dry hands
- A pleasant smell

+ Product appearance:

Clear blue gel

+ Active ingredients:

Ethanol, propanol, Glycerin, Benzalkonium chloride, Carbomer, EDTA, Fragrance, Deionized water

+ Direction for use:

Apply about 3 milliliter of the product on the palm hand on dry state and scrub it for 60 seconds.

To disinfect the hands before the surgery scrub around 5 milliliter of the product for 60 seconds on the hands.

For disinfection of hands before the surgeries, pay attention to the disinfection of the hands to elbow.

+ Warnings:

- For external use only – apply on uninjured skin
- Avoid contact with eyes, wounds, mucous membranes
- In case of eye contact immediately wash eyes with water and call a doctor.
- This product is flammable, keep away from fire and flame. Avoid spraying it on the flame and hot object.
- Keep out of reach of children and food.

+ Packing:

Fast-acting surface disinfectant solution in polyethylene containers are available in 60, 100, 250, 350 and 500 Milliliter and 1, 5, 10, 20 Liter volumes.


+ Other information:

Storage conditions: Store between 5-35 °C in closed container and keep away from the direct sunlight, flammable materials and heat.



Naghsh-e-Jobbe Pars

Health for all- Peroxan, Messenger of Health

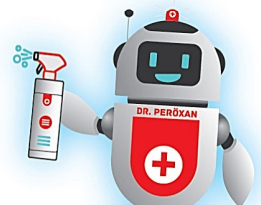



**NON
ALCOHOLIC
HAND
ANTISEPTIC**



**WITH
VITAMIN E
AND
SOFTNER**



**Kills up to %99.99
of Viruses,
Bacteria and Fungi**



► GMP
Universal GmbH UNICERT

► CE
European conformity

► ISO 9001:2015
Quality management

► ISO 22000:2005

► ISO 14001:2004
Environment management standard

► OHSAS 18001:2008
Occupational health & safety
advisory services

► ISO 10002:2004
Quality management -customer
satisfaction

Naghsh-e-jobbe Pars Co. IRAN-Semnan
Tel: +98 23 33653459 Fax: +98 23 33653487
SMS Customer Service:10000822391815
www.peroxan.com Email:njpc@yahoo.com

DAS HA[®]

NON-ALCOHOLIC HAND ANTISEPTIC

**Ready-to-use non-alcoholic hand antiseptic
solution with softener without need to rinse**

+ Uses:

Disinfectant solution with a broad spectrum of antimicrobial effective on gram-positive and gram-negative bacteria, Mycobacteria, fungi, molds and coated and uncoated viruses.

+ Benefits:

- Fast effect without need to rinse
- Compatible with hand skin with persistent and long-term effect on the hands
- With special formulation and has quaternary ammonium compounds with softener and moisturizing materials to increase effectiveness and prevent dry hands
- A pleasant smell

+ Product appearance:

Clear colorless liquid

+ Active ingredients:

Silver ion, Glycerine, Benzalkonium chloride, Sorbitol, Vitamin E, EDTA, Fragrance, Deionized water

+ Direction for use:

Apply about 3 milliliter of the product on the palm hand on dry state and scrub it for 60 seconds.
To disinfect the hands before the surgery scrub around 5 milliliter of the product for 60 seconds on the hands.

For disinfection of hands before the surgeries, pay attention to the disinfection of the hands to elbow.

+ Warnings:

- For external use only, and apply on uninjured skin
- Avoid contact with eyes, wounds, mucous membranes
- In case of eye contact immediately wash eyes with water and call a doctor.
- Keep out of reach of children and food.

+ Packing:

Fast-acting surface disinfectant solution in polyethylene containers are available in 60, 100, 250, 350 and 500 Milliliter and 1, 5, 10, 20 Liter volumes.

+ Other information:

Storage conditions: Store between 5-35 °C in closed container and keep away from the direct sunlight, flammable materials and heat.



Naghsh-e-Jobbe Pars

Health for all- Peroxan, Messenger of Health





CONCENTRATED
**MEDICAL
INSTRUMENT
SANITIZER**



**Kills up to %99.99
of Viruses,
Bacteria and Fungi**



► **GMP**
Universal GmbH UNICERT

► **CE**
European conformity

► **ISO 9001:2015**
Quality management

► **ISO 22000:2005**

► **ISO 14001:2004**
Environment management standard

► **OHSAS 18001:2008**
Occupational health & safety
advisory services

► **ISO 10002:2004**
Quality management -customer
satisfaction

Naghsh-e-jobbe Pars Co. IRAN-Semnan
Tel: +98 23 33653459 Fax: +98 23 33653487
SMS Customer Service:10000822391815
www.peroxan.com Email:njpc@yahoo.com

TETRAMO I[®]

INSTRUMENT SANITIZER

MEDICAL INSTRUMENT SANITIZER

**Concentrated disinfectant and cleaning solution
for disinfection and cleaning medical
and dental tools**

+ Uses:

Concentrated disinfectant and cleansing solution with a broad spectrum of antimicrobial effective on gram-positive and gram-negative bacteria, Mycobacteria, fungi, molds and coated and uncoated viruses.

This product is soluble in water with different percentages in hospitals, clinics, doctors 'and dentists' offices for disinfection and cleaning of all medical, dental and anesthesia instruments, polymer parts and medical equipment. Also it can use for disinfection of tools before autoclaving in the CSSD part or in Ultrasonic device.

+ Benefits:

- Seven days shelf life after dilution of the solution
- Has simultaneous disinfecting and cleaning function
- Appropriate for medical, dental and laboratory equipment for autoclave
- Contains anti-corrosion materials to protect tools against corrosion
- Can be used in ultrasonic devices

+ Product appearance:

Clear yellow liquid

+ Active ingredients:

Didecylidimethylammonium chloride, Benzalkonium chloride, n-(3-Aminopropyl)glycine, EDTA, Distilled water

+ Direction for use:

Wear appropriate gloves before use, and prepare the solution in the following recommended concentrations. Immerse the tools in the solution according to the concentration and contact time and after that brush them. At the end of the immersion time, rinse the tools with running water.

In the 2% dilution for 30 minutes

In the 1% dilution for 60 minutes

+ Warnings:

- It is better to use gloves, masks and suitable cover in order to prevent possible injuries while working with all chemicals.
- Avoid contact with eyes, wounds, mucous membranes.
- In case of eye contact immediately wash eyes with water and call a doctor.
- Keep out of reach of children and food.

+ Packing:

Concentrated disinfectant and cleaning solution for disinfection and cleaning tools in polyethylene containers are available in volumes of 1, 5, 10 and 20 liters

+ Other information:

Storage conditions: Store between 5-35°C in closed container and keep away from the direct sunlight, flammable materials and heat.



Naghsh-e-Jobbe Pars

Health for all- Peroxan, Messenger of Health





**MINOR
SURGERIES
DISINFECTANT**



**Fast-acting
disinfectant**



**Kills up to %99.99
of Viruses,
Bacteria and Fungi**



► **GMP**
Universal GmbH UNICERT

► **CE**
European conformity

► **ISO 9001:2015**
Quality management

► **ISO 22000:2005**

► **ISO 14001:2004**
Environment management standard

► **OHSAS 18001:2008**
Occupational health & safety
advisory services

► **ISO 10002:2004**
Quality management -customer
satisfaction

Naghsh-e-jobbe Pars Co. IRAN-Semnan
Tel: +98 23 33653459 Fax: +98 23 33653487
SMS Customer Service:10000822391815
www.peroxan.com Email:njpc@yahoo.com

DAS S[®]

MEDICAL SURFACE SANITIZER

MINOR SURGERIES DISINFECTANT

+ **Uses:**

This product is ready-to-use solution and spray for skin disinfection for small surgeries in all health centers and has a broad spectrum of antimicrobial activity for a variety of gram-positive and gram-negative mycobacteria, fungi, coated and uncoated viruses.

+ **Benefits:**

- Contains quaternary compounds of ammonium and various alcohols to accelerate persistence and increase antimicrobial properties
- Fast bactericidal
- A suitable alternative to alcoholic cotton during small surgeries

+ **Product appearance:**

Clear red liquid

+ **Active ingredients:**

Ethanol, Isopropanol, Benzalkonium chloride, Chlorhexidine, Deionized water

+ **Direction for use:**

Before any small surgeries, for disinfection the injection surfaces, Spray the solution on the area and wait for about 60 seconds to dry.

+ **Warnings:**

- For external use only – apply on uninjured skin
- Avoid contact with eyes, wounds, mucous membranes
- In case of eye contact immediately wash eyes with water and call a doctor.
- This product is flammable, keep away from fire and flame. Avoid spraying it on the flame and hot object.
- Keep out of reach of children and food.

+ **Packing:**


Fast-acting surface disinfectant solution in polyethylene containers are available in 60, 100, 250, 350 and 500 Milliliter and 1, 5, 10, 20 Liter volumes.

+ **Other information:**

Storage conditions: Store between 5-35 °C in closed container and keep away from the direct sunlight, flammable materials and heat.


PERÖXAN[®]
Naghsh-e-Jobbe Pars

Health for all- Peroxan, Messenger of Health

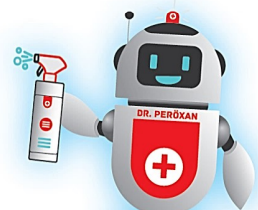



Hemodialysis Machines Disinfection



Dialysis
Machine



**Kills up to %99.99
of Viruses,
Bacteria and Fungi**



UNICERT
GMP



► GMP

Universal GmbH UNICERT

► CE

European conformity

► ISO 9001:2015

Quality management

► ISO 22000:2005

► ISO 14001:2004

Environment management standard

► OHSAS 18001:2008

Occupational health & safety
advisory services

► ISO 10002:2004

Quality management -customer
satisfaction

Naghsh-e-jobbe Pars Co. IRAN-Semnan

Tel: +98 23 33653459 Fax: +98 23 33653487

SMS Customer Service:10000822391815

www.peroxan.com Email:njpc@yahoo.com

SWD 20%[®]

DISINFECTION OF HEMODIALYSIS
MACHINES

HEMODIALYSIS MACHINES DISINFECTION

**disinfectant solution based on
citric acid for disinfection, descaling and hot
washing of all types of hemodialysis machines**

+ Uses:

- Strong hot disinfection of dialysis machines
- Ready-to-use disinfectant solution without the need for dilution
- Has a wide range of effects in killing bacteria, fungi and spores according to the temperature conditions of the manufacturer's instructions
- Without any hazardous materials
- Has descaling properties in addition to a strong antiseptic effect

+ Benefits:

Can be used to disinfect materials such as stainless steel, plastics, types of polymers such as silicon, polyurethane and teflon

+ Product appearance:

Clear colorless liquid

+ Active ingredients:

Citric acid, Didecyldimethylammonium chloride, Benzalkonium chloride, EDTA, Deionized water

+ Direction for use:

Method of use is according to the instructions of the manufacturer of dialysis machines

+ Warnings:

- Avoid contact with skin and eyes, in case of contact immediately wash skin and eyes with water and see a doctor.
- Keep out of reach of children and food.
- Keep away from sunlight, heat sources and high temperature

+ Packing:

Fast-acting surface disinfectant solution in polyethylene containers are available in 250, 350 and 500 Milliliter and 1, 5, 10, 20 Liter volumes.

+ Other information:

- Storage conditions: Store between 5-35 °C in closed container



Naghsh-e-Jobbe Pars

Health for all- Peroxan, Messenger of Health





ALCOHOLIC
HAND
ANTISEPTIC



WITH
VITAMIN D
AND
SOFTNER



**Kills up to %99.99
of Viruses,
Bacteria and Fungi**



▶ ISO 22000:2005

▶ ISO 14001:2004

Environment management standard

▶ OHSAS 18001:2008

Occupational health & safety
advisory services

▶ ISO 10002:2004

Quality management -customer
satisfaction

▶ GMP

Universal GmbH UNICERT

▶ CE

European conformity

▶ ISO 9001:2015

Quality management

Naghsh-e-jobbe Pars Co. IRAN-Semnan

Tel: +98 23 33653459 Fax: +98 23 33653487

SMS Customer Service:10000822391815

www.peroxan.com Email:njpc@yahoo.com

DAS H[®]

ALCOHOLIC ANTISEPTIC HAND

**Ready-to-use alcoholic hand antiseptic solution
with softener without need to rinse**

+ Uses:

This product has a broad spectrum of antimicrobial activity for a variety of gram-positive and gram-negative mycobacteria, fungi, coated and uncoated viruses.

+ Benefits:

- Fast effect without need to rinse
- Compatible with hand skin with persistent effect on the hands
- With special formulation and has quaternary ammonium compounds with vitamin D3, conditioner and moisturizing materials to increase effectiveness and prevent dry hands

+ A pleasant smell

+ Product appearance:

Clear blue liquid

+ Active ingredients:

Ethanol, Isopropanol, Glycerin, Benzalkonium chloride, EDTA, Vitamin D3, Fragrance, Deionized water.

+ Direction for use:

Apply about 3 milliliter of the product on the palm hand on dry state and scrub it for 60 seconds.

To disinfect the hands before the surgery scrub around 5 milliliter of the product for 60 seconds on the hands.

For disinfection of hands before the surgeries, pay attention to the disinfection of the hands to elbow.

+ Warnings:

- For external use only – apply on uninjured skin
- Avoid contact with eyes, wounds, mucous membranes
- In case of eye contact immediately wash eyes with water and call a doctor.
- This product is flammable, keep away from fire and flame. Avoid spraying it on the flame and hot object.
- Keep out of reach of children and food.

+ Packing:

Fast-acting surface disinfectant solution in polyethylene containers are available in 60, 100, 250, 350 and 500 Milliliter and 1, 5, 10, 20 Liter volumes.

+ Other information:

Storage conditions: Store between 5-35 °C in closed container and keep away from the direct sunlight, flammable materials and heat.



Naghsh-e-Jobbe Pars

Health for all-Peroxan,Massenger of Health





**PREP
DISINFECT**



**Fast-acting
disinfectant**



**Kills up to %99.99
of Viruses,
Bacteria and Fungi**



► ISO 22000:2005

► ISO 14001:2004

Environment management standard

► OHSAS 18001:2008

Occupational health & safety
advisory services

► ISO 10002:2004

Quality management -customer
satisfaction

► GMP

Universal GmbH UNICERT

► CE

European conformity

► ISO 9001:2015

Quality management

Naghsh-e-jobbe Pars Co. IRAN-Semnan

Tel: +98 23 33653459 Fax: +98 23 33653487

SMS Customer Service:10000822391815

www.peroxan.com Email:njpc@yahoo.com

DAS P[®]

PREP DISINFECT

+ Uses:

This product is a ready-to-use solution and spray at injection site in order to disinfect in all health centers and has a broad spectrum of antimicrobial activity for a variety of gram-positive and gram-negative mycobacteria, fungi, coated and uncoated viruses.

+ Benefits:

- contains quaternary ammonium compounds and various alcohols to accelerate, persistence and increase antimicrobial properties.
- Fast bactericidal
- Suitable alternatives for alcoholic cotton during injections.

+ Product appearance:

Clear colorless liquid

+ Active ingredients:

Ethanol, Isopropanol, Benzalkonium chloride, Deionized water

+ Direction for use:

Before any injection, for disinfection of the injection area, spray the solution and wait around 60 seconds until dry the site.

+ Warnings:

- For external use only – apply on uninjured skin
- Avoid contact with eyes, wounds, mucous membranes
- In case of eye contact immediately wash eyes with water and call a doctor.
- This product is flammable, keep away from fire and flame. Avoid spraying it on the flame and hot object.
- Keep out of reach of children and food.

+ Packing:

Fast-acting surface disinfectant solution in polyethylene containers are available in 60, 100, 250, 350 and 500 Milliliter and 1, 5, 10, 20 Liter volumes.

+ Other information:

Storage conditions: Store between 5-35 °C in closed container and keep away from the direct sunlight, flammable materials and heat.



Health for all- Peroxan, Messenger of Health

